- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

Kopfleuchte KaWe HiLight LED H-800

CE

de	Kopfleuchte KaWe HiLight LED H-800	4
en	Headlamp KaWe HiLight LED H-800	6
fr	Éclairage frontal KaWe HiLight LED H-800	8
it	Caschetto KaWe HiLight LED H-800	10
es	Lámpara frontal KaWe HiLight LED H-800	12
pt	Lanterna frontal de KaWe HiLight LED H-800	14
ru	Светодиодный налобный осветитель Kawe HiLight LED H-800	16

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User's Manual Headlamp KaWe HiLight LED H-800

Dear buyer, we are grateful to you for choosing KaWe products. Our products are of high quality and long-lasting. This KaWe device complies with the provisions of Regulation EU 2017/745 (European Regulation on Medical Devices), whereby it is classified as a medical device of class 1.

Please, read this user's manual attentively and carefully and pay attention to the maintenance recommendations prior to use.

Please, carefully read the operating instructions prior to use.

Application: Headlamp KaWe HiLight LED H-800 shall be operated only by the authorized qualified personnel.

Intended use: Headlamp KaWe HiLight LED H-800 is used as an illuminating device for medical examinations. The headlamp is put on the head during operation in order to provide for optimal illumination of the area under examination. Additionally, the headlamp may be equipped with a binocular loupe to examine the region under magnification. The following parameters can be selected (magnification x operating distance): 2.5 x 340 mm, 2.5 x 420 mm, 3.5 x 340 mm, 3.5 x 420 mm.

Assembly/Disassembly: Headlamp KaWe HiLight LED H-800 shall be connected to KaWe lithium ion battery. There is a cable for this on the right, which is directly connected to the battery for the headrim or to the battery cable for belt fastening. When the headlamp is used with the battery for the head rim, it shall be fastened of its back side. For this purpose, insert the clamping elements of the battery body to the grooves on the back side of the head rim and clamp it with a light movement. In order to disconnect the clamping elements, move them together again to the opposite direction.

The binocular loupe can be fastened on the back in a holder under the lamp body. For this purpose, loosen the fastening screw on the joint of the binocular loupe. After insertion into the holder, fix the fastening screw in the threaded hole on the front side. To regulate the position of binocular loupes, slightly loosen them. Then tighten the fastening screw to reliably fix the binocular loupes in the required position.

Improper use of the device/contraindications for use: Any other use of use outside the above indication shall be considered improper. The manufacturer is not responsible for damage resulting from such use. Only the user is responsible for the risk. For power supply, use only the battery in the delivery package or the original battery by KaWe.

Commissioning/Handling:

1. Connect the headlamp to the battery for the head rim or belt fastening.
2. To switch the headlamp on, press and hold the control button for approximately one second. The current charge of the battery is displayed

by short flashing of LED indicators. If none of the LED indicators flashes, the battery is completely discharged.

3. After switching on the brightness is at the minimum. To increase the brightness, turn the control knob clockwise. To decrease the brightness, turn the control knob counterclockwise. The brightness is indicated by LED indicators.

4. To switch off, press and hold the control knob for approximately one second.

5. When putting the headlamp on, make sure it fits tight.

Charging of batteries:

1. To charge the batteries of the headlamp, use only a special power unit with an integrated plug included into the delivery package.

2. When the green LED is on at the power unit, the connected battery is charging. During charging, LED indicators on the respective battery remain dark.

3. The green LED at the network power unit goes off, when the battery is fully charged.

4. The headlamp shall not be operated during charging. In this condition, the battery shall not be switched on.

Technical data:

Group (acc. to IEC 62471)	2
Permissible ambient temperature	+10 °C to +40 °C
Relative humidity	10 % to 75 %
Storage temperature	+5 °C to +60 °C
Atmospheric pressure	700 to 1060 hPa
Protection rating	IP X0
Input voltage (network power supply unit)	100-240 V alternating current, 50/60 Hz
Output voltage (network power supply unit)	8.4 VDC
Capacity	Belt battery: 3 000 mAh Head rim battery: 1,500 mAh
Weight (without the battery)	about 260 g
Duration of the battery operation (head rim)	about 4 hours*
Duration of the battery operation (belt fastening)	about 8.5 hours*
Life span (battery)	at least 500 charging cycles
Life span (LED indicators)	about 50,000 hours

*: Duration of operation at the maximum brightness.
The less the brightness, the more the operating time.

Warnings: The device does not need maintenance. In this connection, the body shall not be opened. Repairs shall be performed only by qualified personnel. Damaged parts shall be rejected or replaced.

Long operation may lead to heating of the lamp and in some cases cause discomfort at skin contact.

Do not direct the device straight to the eyes!

In case of visible damages, remove the plug from the outlet immediately. Avoid further operation and send the device to the repair service. Headlamp HiLight LED H-800 is not intended for operation in explosion hazardous premises and near powerful magnetic fields as well as in operating room (anesthesiology departments). Only KaWe components may be used. It is forbidden to modify the design of the device.

Directives

Transportation and storage: To prevent contamination and dust, store the device only in its packaging before installation.

Preparation for cleaning: No special requirements.

Manual cleaning: The headlamp or battery can be cleaned with a wet, soft and lint-free cloth. Use soapy water or other non-corrosive products for cleaning. Binocular loupe lenses can be carefully wiped with a glass cleanser.

Automatic cleaning: Automatic cleaning is impossible/unnecessary.

Disinfection: The body can be wiped with a disinfectant for surfaces. The surfaces must be wiped dry.

Control and functional check: Check the moving parts, for example, joints. Turning the product on. All products: Visually check for damages and wear.

Package: Delivered in a cardboard package with sections cut according to the headlamp shape.

Sterilization: Sterilization is not possible/not required.

Additional information: Follow the operation manual. If you have any additional questions, please, feel free to contact the manufacturer prior to commissioning.

Manufacturer: **KIRCHNER & WILHELM** GmbH + Co. KG

Manufacturer's contact details: Dealer's address or phone number: +49-7141-68188-0.

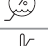
Additional information: The device will serve for years with proper handling and storage.

Warranty: The warranty for the device totals two years from the date of sale with proper handling and following the operation manual. If you have any additional questions or need repair, please, contact your dealer.

General accessories: To find out more about the products, visit our homepage: www.kawemed.com

Basic UDI-DI: 4030155KaWe1201Z5

Symbols:

	Manufacturer
	Factory number, batch
	Please follow the User's Manual
	Article number
	Separate disposal of electric / electronic devices and batteries
	Date of manufacture
	CE conformity mark
	Use the device only in dry rooms
	Protection rating II
	Protect against moisture
	Attention. Electromagnetic field
	Air humidity limits
	Temperature range
	GOST-R certification of export goods imported into Russia
	Medical device

Electromagnetic compatibility of the product: This information can be found at the web site www.kawemed.com

Information for users and patients

All serious incidents involving this device must be immediately reported to the manufacturer and the competent authorities of the respective Member State in which the user and/or patient is located.

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KIRCHNER & WILHELM GmbH + Co. KG