de	Gebrauchsanweisung Praxisliege	2
en	User's Manual Medical Exam Table	5
fr	Mode d'emploi Divan de cabinet médical	8
it	Istruzioni per l'uso Lettino da visita	11
es	Instrucciones de empleo Camilla	14
pt	Manual de operação Maca de consultório	17
ru	Руководство по применению Кушетка медицинская	20

CE

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User's Manual Examination table

Dear Customer, thank you for choosing a KaWe product. Our products are known for high quality and durability. This device complies with the provisions of Regulation EU 2017/745 (European Regulation on Medical Devices), whereby it is classified as a medical product of class I.

Please read the full text of this User's Manual carefully before use and follow the safety instructions.

The manual should be kept neatly next to the table in a place easily accessible for each user.

Intended use: The table is designed for examination, rest and treatment of patients in medical premises.

Rated load: 400 kg static load, 135 kg dynamic load

User: Use must only be carried out by authorized personnel or persons who have been properly instructed.

Target patient group: All who meet the specified load.

Contraindications: Any other use or use outside the above intended use shall be considered improper. The manufacturer is not responsible for damage resulting from such use. In this case, all risks are assumed solely by the user.

-The table **cannot** be used for massage or chiropractic!

Safety instructions:


- The table is designed for examination, treatment of patients, as well as for procedures. Any other use may result in accidents or damage to the product. The table cannot be rearranged. In the event of impermissible interventions, our warranty and the CE certificate will no longer be valid.
- The head section can only be loaded with the patient's head and partly with the upper torso, in no case should the head section be sat on or loaded from the edge with more than 10 kg.
- The table **cannot** be used for massage or chiropractic!
- Observe the table load capacity!
- Do not use the table without the side rails or with the side rails down for resting or waking up, always use two side rails!
- **It is forbidden to equip the table with casters or wheels!**
- Repairs must only be carried out by trained personnel.
- To protect the patient, please use a medical grade upholstery cover.
- If you find unusual changes, extraneous sounds or other hazards during use, please contact us immediately. Further use of the table is prohibited until the causes are identified.
- Observe the rated load: 400 kg static load, 135 kg dynamic load.

Installation and handling


In no case do the pictures of the components **correspond** to the actual dimensions and serve **only** to understand the assembly principle.

Basic instructions: The parts shown here are supplied as a kit only if they have not been pre-installed. Please **never** use electric screwdrivers or ratchet wrenches as these parts are not designed for this and may be damaged. For optimal positioning of parts against each other, all bolts are first screwed in by hand, then finally screwed in a crosswise pattern at the end of installation. If parts have been damaged as a result of improper installation, replacement will only be carried out for a fee.


Parts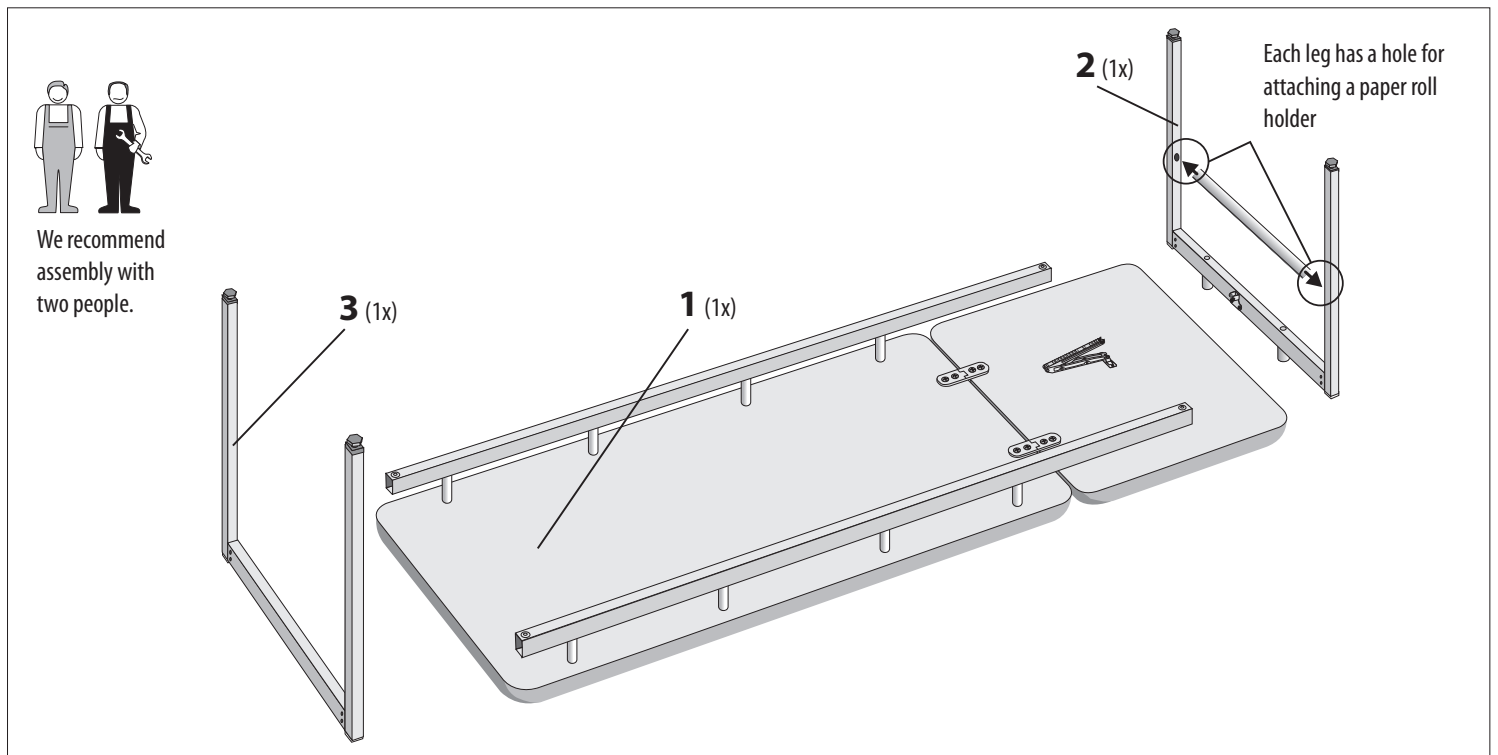

Installation

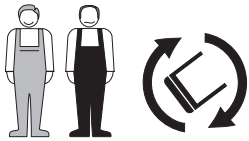
In order to preserve the upholstery, only turn the table over with two people! – Make sure that the head section frame is supported by hand when turning over.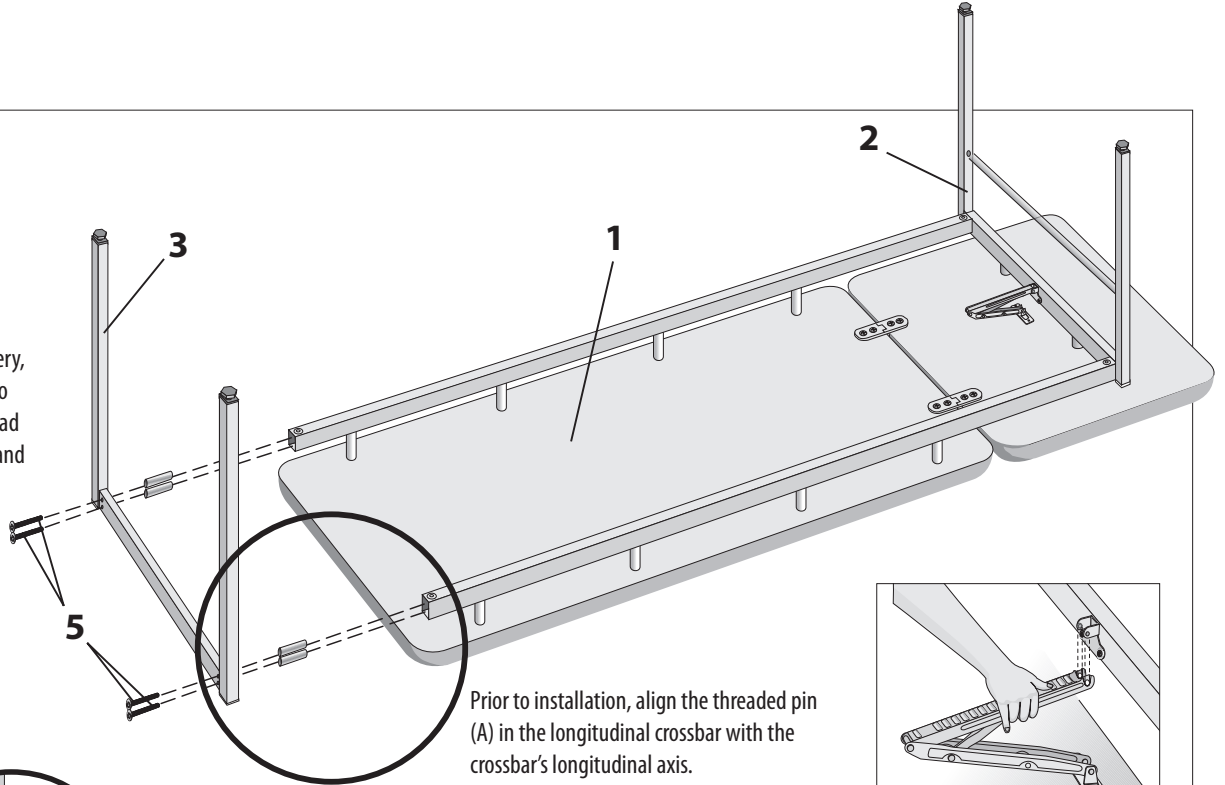


Prior to installation, align the threaded pin (A) in the longitudinal crossbar with the crossbar's longitudinal axis.

After that, first install the groups of legs by hand, then finally tighten them crosswise. In the end, fix the head section lifting mechanism (see figure on the right), while the table must be in the position shown.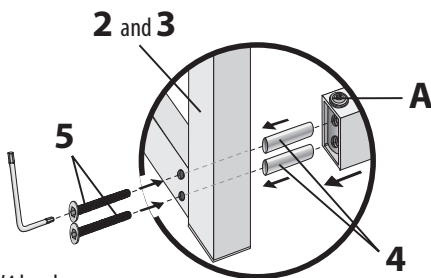


SW4 hex key

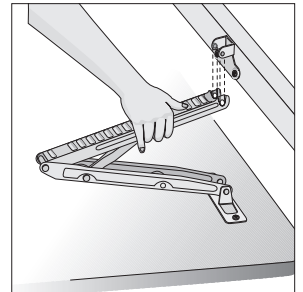The head section lifting mechanism must be pulled up to insert it into the already assembled frame and lock it into place.

Head section inclination angle adjustment


There are 9 possible positions for adjusting the head section inclination angle.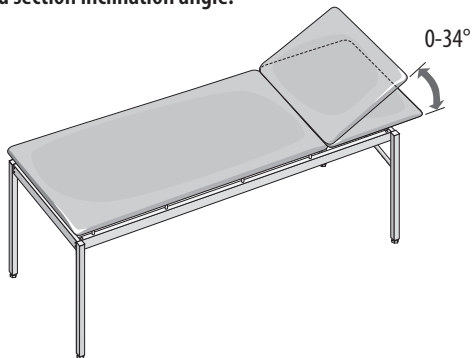

Raise the head section from the table end. Bring the section to the desired angle until it clicks. To lower the head section, it is necessary to raise it again from the end until it stops. Then lower the section to the lowest position.

Now you can set the head section to any other position again. While adjusting, carefully support the head section with your hand. The cushion base protrusion (according to DIN EN ISO 13857) protects the hand or fingers from being pinched.

Paper roll holder installation

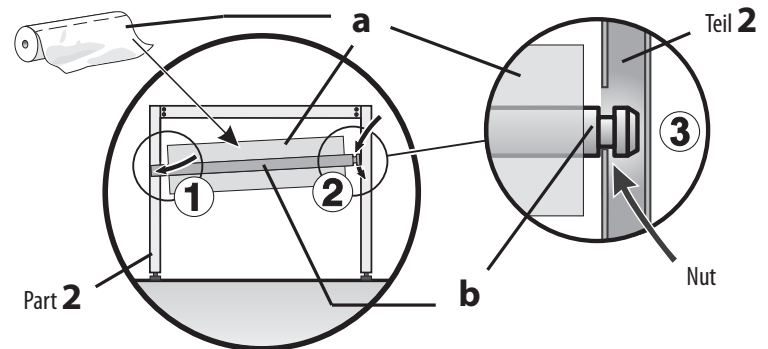- ① The legs of **Part 2** (in the table head section area) in their upper part have two holes located opposite each other. Place the paper roll **a** on the paper roll holder **b** and first insert one end of the holder without a groove into the hole on the leg (see figure).
- ② Then insert the paper roll holder **b** with the other grooved end into the opposite leg hole (see figure), making sure that the groove is inside the leg to prevent popping out.
- ③ Now, if necessary, you can unwind the paper and lay it on the table to protect the patient.

Maintenance / Cleaning / Disinfection:

The table does not require maintenance during operation. However, the user should inspect the table at least every 6 months to ensure that it is in good condition. To do so, all bolted connections must be checked for sufficient tightening! Room temperatures below 7 °C lead to rust on chrome parts! Our surfaces can be wiped with either a dry or wet wipe. In case of wet treatment, wipe the surface dry.

ATTENTION! Do not use a very wet wipe to prevent water from getting inside the structural elements.

Do not use cleaners containing abrasive additives! Such products can damage and scratch surfaces (including chrome-plated ones). Avoid contact of chrome-plated surfaces with halogens such as fluorine, iodine, bromine or chlorine/chloride (e.g., hydrochloric acid) or their derivatives, as this may destroy the chrome and nickel coating. Chrome-plated parts cannot replace high-quality steel in the field of hygiene!

The known bases of biologically active substances in disinfectants lead to surface damage if the product is erroneously selected and used incorrectly. For disinfection, we recommend **“Mikrozyd sensitive Liquid”** (liquid) and **“Mikrozyd sensitive Wipes”** (wipes) from Schülke & Mayr*. Particular attention must be paid to ensure that all active substances are removed from the surface after a short exposure time (1 minute), otherwise the surface (including stainless steel surfaces!) may be damaged due to chloride compounds, especially by corrosion. Do not allow the agent to penetrate structural elements, since an increase in the concentration of active substance residues that occurs in these places can lead to subsequent destruction with loss of the functions of mechanical elements. Therefore, never disinfect by spraying! Please note that after a certain exposure time, all residues of active substances must be removed to prevent further damage. If you do not follow this recommendation, the service life of the table may be significantly reduced. Despite our recommendations for the use of disinfectants, the user is not relieved of the obligation to be critical of the table surface disinfection. If in doubt, contact the disinfectant manufacturer, as only the manufacturer can give an answer about this product's suitability and report any changes made to the active substance formula. Please note that potential damage to the material may become visible only after a long exposure time.

Faux leather upholstery

The upholstery does not require special care to preserve the material. Clean normal dirt on the upholstery with a wet wipe, then wipe with a dry cloth. In case of heavy contamination, clean with grease-free soapy water. Never use solvents or greasy cleaners! Protect the upholstery against sebum, alcohol-based solutions and creams as there is a risk of material damage!

Special care is required when using disinfectants!

*Schülke & Mayr GmbH, Robert-Koch-Str. 2, D-22851 Norderstedt
Fax 49 40 521 00 - 318, www.schuelke.com

Environmental conditions: Room temperature (min. 7 °C)

Disposal: The product shall be disposed of in accordance with local environmental regulations.

Accessories: Chrome-plated paper roll holder for the table, 65 cm wide.

Spare parts: Plugs (set of 4 pcs.), paper roll holder, position lock, hinges (including bolts).

Symbols:

	Manufacturer & date of manufacture		Attention
	Batch number		CE conformity mark
	Please follow the User's Manual		Medical device
	Article number		UDI Data Carrier
	Store in a dry place		

Information for users and patients

All serious incidents involving this device must be immediately reported to the manufacturer and the competent authorities of the respective Member State in which the user and/or patient is located.

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KIRCHNER & WILHELM GmbH + Co. KG